

ATTACHING EAR LOOP ADJUSTERS on

SUBMASKLBL3 | SUBMASKSBL3

These three-layer face masks feature a 100% polyester outer shell that can be decorated with sublimation or heat transfer printing and a cotton inner layer that follows CDC recommendations for cloth facial coverings.

ATTACHING THE RUBBER EAR LOOP ADJUSTERS.

(for smaller faces)



1

Bend a small paperclip as shown here.
Insert into the small end of the rubber slider.




2

Push all the way through.

3

Grab the elastic ear loop and pull through the rubber slider.

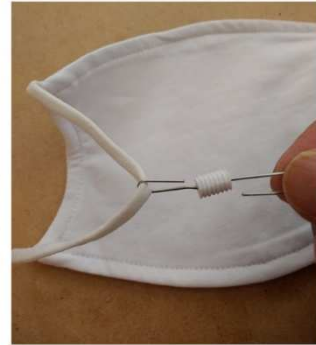

4

This should be your result!

